



Digital



2* 120
Memory



Automatic



One Button
Operation



Arrhythmia
Detection

BPM-107 Digital Blood Pressure Monitor

Features:

- One button operation; just sit and press start for the blood pressure monitor to start.
- Very low noise during reading.
- User memory for 2 people; Storage for 2*120 readings.
- Arrhythmia Detection.
- Compatible with both batteries and adapters.
- Huge Screen to make it easy for everyone to read the blood pressure readings.
- Ergonomic Design.
- Highly Accurate.

Specifications:

Display
Measurement Mode
Measurement Range

Accuracy

Resolution
Power Supply
Storage Temperature and Moisture
Using Temperature and Moisture

Digital display
Oscillometric method
Pressure: 30~280 mmHg
Pulse: 40~199beat/minute
Pressure: ± 3 mmHg (0.4kPa)
Pulse: $\pm 5\%$ of display reading
1mmHg
4* AAA batteries
(-10~+50) °C Moisture: (10~95)% RH
(-5~+40) °C Moisture: (15~80)% RH

How to Use:



Put the cuff on your arm ensuring the arrow on the cuff is in the correct direction.



Make sure that the bottom end of the cuff is 2-3 cm from the elbow joint.



The pipe should be placed in the middle of the arm.



Don't make it too tight or bend the tube.



Place your arms flat on the table with your palms up and your body sitting straight. The center of the cuff should be placed at the same height as your heart position. Please note that the air tube on the cuff is not twisted.

Contact us:



Mobile No./ WhatsApp No. : +91 9310113243



Web : www.klifecare.com



Email : info@klifecare.com

Kannu Impex (India) Pvt. Ltd.



412, Suncity Trade Tower, Sector-21, Gurugram-122015