

HA-101 Hearing Amplifier-101

Features:

- Flexible, 360 degree rotation
- Three sizes plug.
- Soft and comfortably fit.
- 4 Points volume control.
- Noise cancellation and clear sound.

Specifications:

Model Output OSPL Max. FOG50 MAX Gain **EQ Input Noise** Working Current Total Harmonic Distortion Rated Voltage Battery life Frequency Range (Not narrower than)

HA-101 ≤ 133+3dB \leq 37+3dB ≤ 33+3dB ≤ 4.5mA ≤ 7+3% D.C 1.5V LR754 320Hz-3200Hz

How to Use:



Open the cartridge cover, put the battery in correct direction.



Before use, make sure the power is turn off, or make the volume to "1"



Put the ear hook on your ear plug in the ear plug tightly. (Note The unit is on "off" position. And volume is on "1" otherwise the instant sound will make you uncomfortable).



Turn on the power and adjuster the sound volume to get clear sound.



If you hear the whistling sound. please check whether you have plugged in the ear plug tightly, please change the other size plug.

Contact us:



Mobile No./ WhatsApp No. : +91 9310113243



