



Fast Heating



Even Distribution Of Heat



4 Layer Insulation



For Muscular Pain, Cramps & Sprains



Protection From Over Heating



3 Level Temperature Settings

HB-101 Heating Belt

Features:

- Fast Heating
- Even Distribution of Heat
- 4 Layer Insulation
- Protection from Over Heating
- 3 Level Temperature Settings
- For Muscular Pain, Cramps & Sprains



Joint Pain



Sports Injury



Muscle Cramps & Pain




Menstrual Cramps



Muscle Spasm & Inflammation

How to Use:

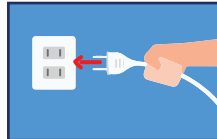


Plug the power cord in the electrical socket and switch it on.




Wait for 3-4 minutes for the Heating Belt HB-101 to heat up.



Place the heating belt on the area to be treated.

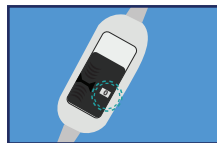

Select the temperature controller setting as per use.

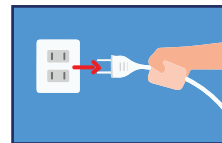

Turn the heating belt off and unplug the power cord after use.

Specifications:

Model
Temperature controller with 3 Heat levels

Maximum Temperature
Power Supply
Wattage

Heating Belt
Position 1 : 47°C - 52°C
Position 2 : 53°C - 62°C
Position 3 : 63°C - 85°C

90°C
AC 220-240V, 50Hz
45 Watts

CAUTION

(If used improperly, this product can cause severe burns to skin and damage to property.)

Contact us:



Mobile No./ WhatsApp No. : +91 9310113243



Web : www.klifecare.com



Email : info@klifecare.com

Kannu Impex (India) Pvt. Ltd.



412, Suncity Trade Tower, Sector-21, Gurugram-122015