



Scan  
Measuring



Fever  
Alarm



Quiet



Ear



Forehead

**Green  
yellow  
Red**

Three Colour  
Indication



## IR-101 Non-Contact Infrared Thermometer

### Features:

- Non-Contact detection
- Body or object temperature detection
- Displays temperature reading in seconds
- Sound can be turned on/off
- Easily switches between °C and °F
- Gives an auditory and visual Warning
- Warning (three colours LCD display)  
20 readings memory recall BPM and latex free

### Specifications:

Power Supply Mode  
Operating Voltage  
Battery Model  
Display  
Measure time  
Measuring Distance  
Measuring Range  
  
Accuracy  
Resolution  
Memory  
Low-battery Alert  
Automatic Power-off  
Operating Environment  
  
Storage and Transportation

Internal power supply  
DC 3V  
AAA x 2  
LCD  
About 1 second  
1/2"  
Forehead: 34.9°C–42.2°C  
Object: 0.0°C–100.0°C  
±0.4°F/±0.2°C from 94.8°F to 108.0°F  
0.1°C (0.1°F)  
20 temperature readings  
when the power voltage is lower than 2.5 V±0.1V.  
Automatically powers off in 10 ± 1 seconds.  
Temperature: 10°C–40°C  
Humidity: 15%–95% RH, non-condensing  
Temperature: -20°C to 55°C  
Humidity: 0- 95% RH, non-condensing

### How to Use:



Battery Cover

Open the battery cover and put batteries inside the thermometer.



Mode Button

Hold the Mode button for a short while to switch between forehead and object.



Speaker Button

Hold the Speaker button for a short while to switch between Mute and un-mute.



Point the temperature probe at the center of the forehead with a distance between 0.5-2 inches, the temperature will be displayed on the screen instantly.

### Contact us:



Mobile No./ WhatsApp No. : +91 9310113243



Web : [www.klifecare.com](http://www.klifecare.com)



Email : [info@klifecare.com](mailto:info@klifecare.com)

Kannu Impex (India) Pvt. Ltd.



412, Suncity Trade Tower, Sector-21, Gurugram-122015