

NEB-107 Nebulizer

Features:

- Strong built, easy to use and carry, all models with handles
- → High performance nebulizers
- Fit for both adults and children
- Low noise production

Specifications:

Medication Capacity Sound Level Nebulization Rate Flow Rate Operating Temperature

Operating Humidity

10 ~ 20 mL ≥ 60 dbA 0.2 mL/minute 3.5-6 Litre/minute 10°C ~ 40°C

10% ~ 95% RH

How to Use:



Place the medication into the nebulizer Chamber.



Connect the tubing to machine and to the medicine chamber.



attach the mask.



Turn on the air compressor and use the nebulizer.



Continue to inhale the medication. Sit up and keep inhaling the medication until the mist stops.



Turn off the nebulizer and wash the medicine cup and mouthpiece with warm soapy water and rinse them with water or towel.

Contact us:





