

NEB-108 Nebulizer

Features:

- Strong built, easy to use and carry, all models with handles
- → High performance nebulizers
- Fit for both adults and children
- Low noise production

Specifications:

Medication Capacity
Sound Level
Nebulization Rate
Flow Rate
Operating Temperature

Operating Temperature
Operating Humidity

10 ~ 20 mL ≤ 60 dbA

0.2 mL/minute

3.5-6 Litre/minute

10°C ~ 40°C

10% ~ 95% RH

How to Use:



Place the medication into the nebulizer Chamber.



Connect the tubing to machine and to the medicine chamber.



Close the cap of chamber attach the mask.



Turn on the air compresso and use the nebulizer.



continue to inhale the medication. Sit up and keep inhaling the medication until the mist stops.



Turn off the nebulizer and wash the medicine cup and mouthpiece with warm soapy water and rinse them with water or towel.

Contact us:





