



Tuberculosis(TB)-IgG antibody

Catalog No.:BG401C BG402S

NAME AND INTENDED USE

Rapid Gold Immuno-assay Kit for Antibody IgG to Tuberculosis (TB) is an in vitro immunoassay for the qualitative determination of Anti-TB antibody IgG in human serum or plasma in one step. It is for diagnosis of early infection and epidemic survey.

PRINCIPLE

The kit uses one step method to detect anti-TB antibodies IgG in serum or plasma. The purified specific recombinant TB antigen is immobilized on the nitrocellulose membrane at the test line region and another specific TB Ag conjugated to the colloidal gold is coupled on the label pad. To perform the test, a sample is added to the sample well of the test card. The sample flows through to the label pad. If the sample contains Antibody IgG to Tuberculosis, the antibody will bind to the antigen coated on the colloidal gold particles to form antibody-antigen-gold complexes. These complexes move on the nitrocellulose membrane by capillary action toward the test line region, and react with specific recombinant TB antigen to produce a pink color on the membrane. A second red control line will always appear in the result window to indicate that the test has been correctly performed and the test device functions properly. If Antibody IgG to Tuberculosis (TB) is not present or lower than the detection limit of the test, only the control line will be visible. If the control line does not develop, the test is invalid. The pink color intensity will reflect the antibody intensity in the sample. The test is fast and easy to operate.

MATERIALS PROVIDED

Antibody IgG to Tuberculosis test card/strip
Each cassette/strip contains a test strip with TB antigen on the test region of the membrane and colored Tuberculosis antigen-gold conjugate pad.

SAMPLE COLLECTION AND PRESERVATION

Blood serum samples are routinely prepared from vein. Blood plasma samples are routinely prepared with routine amount of anticoagulant such as heparin or sodium citrate. Samples may be stored at 2-8 °C within five days without interfering with the assay performance. For long-term storage of specimens, -20 °C or colder is recommended. Repeated freezing and thawing of specimens is not recommended and may cause erroneous results. Samples with cloud or precipitation should be centrifuged or filtered before test. Prevent serum from bacteria contamination during collection and storage.

TEST PROCEDURE

1. Bring Membrane Plate and samples to room temperature (20-30 °C) before use (approximately 30 minutes).

2. Remove the test cassette/strip from the sealed foil pouch and place the plate on flat desk.





3. Deliver 2 drops (80 -100 µL) of the sample to the sample well/sample pad.
4. Read the result between 15-20 minutes. A strong positive sample may show result earlier.

Note: Add 1 drop of Saline or Phosphate-Saline buffer (common buffers used in clinic not provided in the kit) into the sample well if flow migration is not observed within 30 seconds in the result window, which could occur with a highly viscous specimen.

INTERPRETATION OF RESULTS

- Positive: Two pink lines appear in both C and T position
- Negative: Only one pink line appear in C position
- Invalid: The control line next to the test line does not become visible within 20 minutes after the addition of the sample.

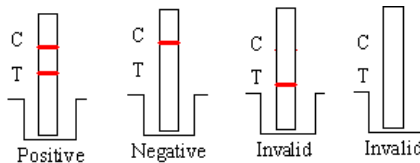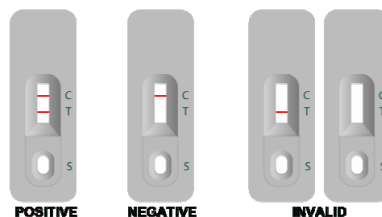

PERFORMANCE CHARACTERISTICS

IgG ELISA Test	TB IgG Rapid Test		Total
	Positive	Negative	
Positive	348	0	348
Negative	2	675	677
Total	350	375	1025

Relative Sensitivity: 100%,
Relative Specificity: 99.7%,
Overall Agreement: 99.8%

LIMITATION

The testing is for qualitative and assistant diagnosis. Confirmation of infection should refer to the clinical and other diagnosis.

PRECAUTIONS

1. Both plate and sample should be with room temperature 20-25 °C
2. Test sample must be fresh.
3. The result after 20 minutes is invalid.
4. Diluted sample may generate false positive result
5. Bacteria contamination, serious hemolysis, jaundice and high lipid blood will cause wrong result. The sample should be re-collected.
6. Wrong result might be caused by expired kit or problematic samples.

PACKAGE SIZE

40 tests/Kit

STORAGE AND STABILITY

Room Temperature.

EXPIRATION

The shelf life is 24 months. Do not use the kit beyond its expiration date.